

FLEXIBLE treatment for **PERIPHERAL** thrombus



Tru-Seal® Hub
Easier Guidewire Management



Flexible 120cm Shaft
Optimal Length for a Wide Range
of Peripheral Interventions



Combination Therapy
Power Pulse™ Delivery Option for
Tough Peripheral Thrombus

AngioJet® Xpeedior® Thrombectomy Catheter


High-speed saline jets inside the Xpeedior catheter create a powerful low pressure zone that pulls thrombus into the catheter and removes it from the body. Cross-Stream® technology can treat larger arteries without vessel wall contact.

Proven safe in thousands of procedures, the Xpeedior catheter is compatible with 8FR guides and 0.035 guidewires so it works with the standards of your practice.


Tru-Seal® Hub

Tru-Seal hub provides easier guide wire management with a simple click. Tru-Seal is easier to use and offers better hemostasis than old screw-close valves.

Push – it's open...pull – it's closed, and sealed.

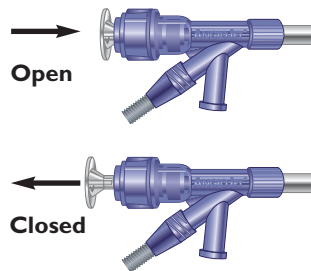

Mechanism of Action

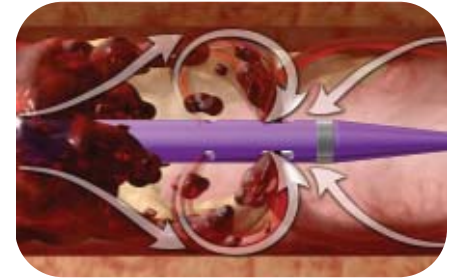

Patented Cross-Stream action utilizes complex fluid flow patterns to capture and remove thrombus from even large vessels.


Xpeedior with Power Pulse™ Delivery

The One-Two Punch combination for tough peripheral thrombus

Power Pulse Delivery used with the AngioJet Thrombectomy System provides a single-catheter option for faster and more thorough removal of tough peripheral thrombus.

Combining two proven therapies — mechanical thrombectomy and pharmacological thrombolysis. Power Pulse Delivery provides the benefits of each therapy with the convenience of a single delivery system.

How Power Pulse Delivery Works

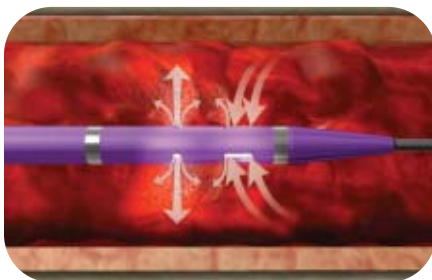

Power Pulse Delivery enables infusion of medication directly into the clot, saturating the thrombus. The action of Power Pulse Delivery helps disrupt difficult thrombus and prepare it for rapid removal using the Xpeedior catheter.

The Power Pulse Kit is designed to work with the AngioJet System and allows you to easily switch between AngioJet Thrombectomy and Power Pulse Delivery.

Catheter Specifications

Min. Vessel Diameter	≥ 3mm
Low Profile Tip	2.0-1.2mm
Working Length	120cm
Max. Shaft Diameter	6F
Double Marker Band	15 mm
Guidewire Compatibility	OTW 0.035"
Guide Compatibility	8F ≥ 0.086"
Sheath Compatibility	6F

Order Information

AngioJet Ultra System

Xpeedior Thrombectomy Set	104307-001
Ultra Power Pulse Kit	106903-001

AngioJet Series 3000 System

Xpeedior Catheter	104307-001
Pump Set	102184-001
Power Pulse Kit	104834-001

MEDRAD | **POSSIS**
Interventional

MEDRAD Interventional-U.S.A.
One Medrad Drive
Indianola, PA 15051-0780 USA
Phone: 1 800 MEDRAD1
1-800-633-7231