

Configuration File Information for the MEDRAD 1.5T Breast Array Coil

11 X Software

The Configuration Files for the MEDRAD 1.5T Breast Coil are located in the systems Configuration File Manager. Please locate the following files in the available coil list:

BreastPA This file turns on *both* sides of the coil
BreastPA Right This file turns on *only* the right side of the coil
BreastPA Left This file turns on *only* the left side of the coil

Click and Add each file name to the site list.

A system re-boot will be necessary following the addition of these file names.

Please Note:

The MEDRAD Breast Array coil is **NOT** compatible with Vibrant

The ability to scan with the left only and right only side of the coil is only available at the Excite platform.

Please call 1800 MRCOILS with any questions