

Gore Cardiovascular Array Configuration File Information

11x Software

The Gore Cardiovascular Array Configuration File is located in the system's Configuration File Manager. Please located the following file name in the Available Coil List:

CARDIOVASCULAR

Click and Add this file to the site coil list. A system re-boot is necessary following the addition of this file.

Please Note:

The **CARDIOVASCULAR** file should be selected regardless of which combiner box is in use with this coil. This coil comes with 3 combiner boxes. This coil and the 3-combiner boxes can be used on 4 or 8 channel systems. Ensure the box in use is properly connected to the cable, and properly connected to the phased array port.

Please refer to the technical tip information for this coil also located on the product page for this coil, here at www.medrad.com

Please call **1800 MRCOILS** with any questions



Picture of all 3 combiner boxes