

**Configuration File Data -
M42EA 1.0T Extremity Array
Non-Excite Systems**

	LGEXTPA	SMEXTPA	LOEXTPA*
Coil Name	LGEXTPA	SMEXTPA	LOEXTPA
Coil Type	1 [this is a head coil]	1 [this is a head coil]	1 [this is a head coil]
Extremity Coil	no	no	no
Cable Loss	Same as the Signa Head Coil Cable Loss	Same as the Signa Head Coil Cable Loss	Same as the Signa Head Coil Cable Loss
Coil Loss	Same as the Signa Head Coil Coil Loss	Same as the Signa Head Coil Coil Loss	Same as the Signa Head Coil Coil Loss
Recon Scale Factor	1.0.	1.0.	1.0
Linear vs. Quad	1	1	1
Multiple Receiver Coil?	yes	yes	yes
Number of Receivers	4	4	1
Starting Receiver ID	0	0	0
Ending Receiver ID	3	3	0
Multi-Coil Port Enable	15	15	15
Multi-Coil Error Enable	15	15	15
Additional Transmit Attenuation	30 (3.0 dB)	80 (8.0 dB)	60 (6.0 dB)
Number of Fast Receivers	0	0	0
Starting Fast Receiver ID	4	4	4
Ending Fast Receiver ID	4	4	4
Fast TG Start TA	110	110	100
Fast TG Start RG	5 (5X) Systems 11 (LX 8X)	5 (5X) Systems 11 (LX 8X)	5 (5X) Systems 11 (LX 8X)
Multi Coil Recon Enable	0	0	14
Head Default Freq Dir	1	1	1
AutoShimRcvr	0	0	
SCIC++parameters for axial plane	0	0	
SCIC++parameters for sagittal plane	0	0	
SCIC++parameters for coronal plane	0	0	
mcSwitch select	0	0	
Multi-coil separation direction	2	2	

* - Not needed for versions of 8.X software that include the AutoShimRcvr parameter.