

11 X Excite Configuration Information for the Medrad 1.5 Tesla T/R PA Extremity Array Coil

Please note: Coil ID chips are required for ALL transmit/receive coils at this sw platform. Failure to have the chip id in place will prohibit the coil from scanning.

If your coil is in need of a Chip Id Upgrade, please contact the Medrad Service department at 1800 633-7231 Ext 5100

Please note: The Medrad configuration files are **pre-loaded** in this software platform. The files are located in the Available Coil List in the Configuration File Manager. The files should be added to the Site Coil List, Saved and followed by a system re-boot.

For assistance with configuration files, please contact your Field Service Engineer or call 1-800-MRCOILS (672-6457)

There are 2 Configuration Files for this coil:

LGEXTPA	Use for knees, elbows, feet and ankles.
SMEXTPA	Use for smaller loads, toes, wrists, fingers etc.

**For technical questions please contact Coil Technical Support at
1800 MRCOILS**

