

MedRad COILS

Gore Torso Array 1.5T Configuration File

Please be aware that some of the terminology may be different depending on the software level.

Coil Name	TORSOPA
Coil Type	3
Extremity Coil	no
Cable Loss	Same as the Signa Body Coil Cable Loss
Coil Loss	Same as the Signa Body Coil Coil Loss
Recon Scale Factor	Prior to 8.2 the value should be 1.0 times the numerical value of the head coil. At 8.2.5 and beyond, use 1.0.
Linear vs. Quad	1
Multiple Receiver Coil?	yes
Number of Receivers	4
Starting Receiver ID	0
Ending Receiver ID	3
Multi-Coil Port Enable	15
Multi-Coil Error Enable	15
Additional Transmit Attenuation	0
Number of Fast Receivers	0
Starting Fast Receiver ID	4
Ending Fast Receiver ID	4
Fast TG Start TA	120
Fast TG Start RG	5 (5X systems) 11 (8X systems)
Multi Coil Recon Enable	0
Head Default Freq Dir	0

Any Questions? Please Call 1 - 800 - MRCOILS

