

1.5 T NVA 8 HD Configuration File Information
12 X Hardware and 12x Software
GE Exclusive

You will find the Configuration Files for the MEDRAD NVA 8 HD coil conveniently located in the Available Coil List in the Configuration File Manager's Available Coil List. The files names are listed as:

HDNV Array (Turns on the entire coil)

HDNV Head (Turns on only the head section)

HDNV Neck (Turns only on the neck section, anterior and posterior neck)

HDNV Spectro (Use for 2D and 3D Spectroscopy)

HDNV Head Neck (Turns on the head section and the posterior c spine elements)

The files should be added from the Available Coil list to the Site Coil List.

A system re-boot will be necessary.