

MEDRAD 1.5T Multi-Purpose Array Configuration File Information

GE 8x and 9x software systems

The Medrad Multi-Purpose Configuration files are available in the system's Configuration File Manager. The config file names are as follows:

SHOPA2 *This file is used when the posterior element is removed. This file activates the elements located in the anterior paddle only.*

SHOPA3 This file uses the anterior and posterior elements together.

These files are located in the Available Coil List. They should be added to the site list via the Configuration File Manager. A system re-boot is required following the addition of config files.

Please see the Technical Tip Bulletin Board for positioning tips.

Call **1800 MRCOILS** with any questions.

