

MEDRAD[®] MR Coils

GE 11 X Excite Configuration Information for the Medrad 1.5 T Shoulder Array Coil

Please note: The Medrad configuration files are **pre-loaded** in this software platform. The files are located in the Available Coil List in the Configuration File Manager. The files should be added to the Site Coil List, Saved and followed by a system re-boot.

For assistance with configuration files, please contact your Field Service Engineer or call 1800 MRCOILS

There are 2 files for this coil:

- 1. SHOPA3**
- 2. SHOPA2**

Please consult the technical tip bulletin board located on the Medrad Web page for coil positioning information.

