

MEDRAD NVA 4 Channel Coil Configuration File Information 9.1 – 9.4 Software

The Configuration Files for the Medrad NVA 4 Channel Coil are located in the available coil list. They are listed as follows:

- NVPA** Use this file for C-spines, Carotids, Soft Tissue Necks etc
- HRBRN** Use this file for standard brain imaging including circle of willis
- FASTBRN FR** Use this file for EPI if your system has a fast receiver
- FASTBRN NFR** Use this file for EPI if your system does **not** have a fast receiver

The files should be added from the Available List to the Site List, Saved to the config file list, followed by a system re-boot.

Please call 1800 MRCOILS for MR Coil Technical Support